

# Installation Instructions

H2Otap 3000 - H2Omtap 3000 - H2Ower 3500 - H2Ower 3600

Biogen Technologies filters are high quality products, produced to highly meet Sanitary Standards requested.



Biogen Technologies S.L.  
Experts in water hygiene

Please take into account carefully instructions:



## Storage

Keep the filters in clean, dry conditions and with its original Package.

Never open the filters packaging time before its installation and usage.

Please don't use a filter with a damaged packaged or if it has been opened time before.



## Installation

Please pay special attention to the necessities before entering a controlled environment. Such us ICU (unit of intensive care), operating rooms, etc. Places in which it would be recommended to use: surgical gown, suitable bonnets, masks and sterile gloves, etc.



## Usage Conditions

2 months of use.



1 Clean and 2 disinfect hands. We recommend the use of sterile gloves.



2 Use an individual bag for removing the filter to be changed.



3 For removing the filter you 4 have to press the bottom in the conector, please see following image, and at the same time stretch slowly the filter downwards.



4 Open the packaging from above, without touching the filter.



5 Place the filter as it shows in the following image, when you instal it correctly you will listen a "click" sound.



6 Remove the 7 filter's packaging once installed.



7 Open the tap so to check that water flows correctly, in case there is a problem this means it is installed incorrectly, please follow again the mentioned steps.



8 Place sticker with installation 9 date so to control changes.



9 In order to avoid retrograde contamination please change the filter in the indicated date.



10 Highly advised to clean and disinfect again the hands.

V.03-19



**Biogen Technologies S.L.**  
C. Juan Ramón Jiménez, 1  
08960 - Sant Just Desvern  
Barcelona - España

**For more information:**  
[www.biogenfilters.com](http://www.biogenfilters.com)  
[info@biogenfilters.com](mailto:info@biogenfilters.com)  
+34 93 372 65 84